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FIELD OF USE:

The PODOSCOPE 6066 is an appliance for studying static footprints using a mirror and tangential LED lighting.

WARNING

Your PODOBAROSCOPE 6068 was designed and validated to carry, under normal use, loads of up to 150KG

Before proposing to your patient to step onto your PODOBARSCOPE 6068, you must ensure that the total weight of the patient is not more than 150KG.

PODOBAROSCOPE 6068 is designed to take static loads, it must not be subjected to stresses due to movements other than those linked to the patient stepping on and off the appliance

PODOBAROSCOPE 6068 contains electrical components, it must only be connected to the electrical network with the transformer and the power cable supplied, or a strict equivalence (see technical characteristics at the end of the manual).

PODOBAROSCOPE 6068 can not be modified or opened, except by professionals authorized by our department.

DO NOT USE CLEANING PRODUCTS WITH ALCOHOL TO CLEAN UP THE DEVICE.

If, despite all the care we have taken in the design and manufacture of your podobaroscope, you discover a defect, or if in doubt about its use, we please contact our service department by phone at +33 385 779 077 or by mail at : sav@capronpodologie.com

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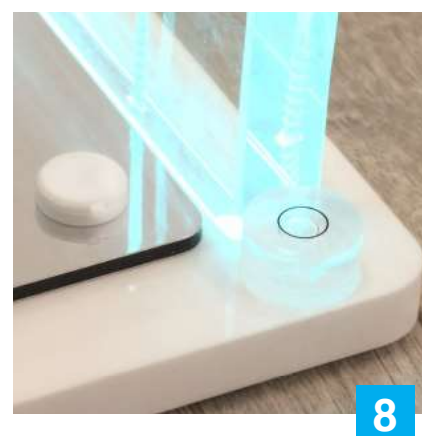
Medical device class : Class I

CE medical device.

Complies with European directives:

- 2006/95/CE of december 12th 2006 on low voltage devices.
- 2004/108/CE of december 2004 on the EMC
- Rule UE 2017 / 745

The information provided in this document is subject to change without notice and the photos are noncontractual.

- 1 «U» - Acrylic glass
- 2 Base
- 3 Mirror
- 4 Remote control
- 5 Power supply 220 Vac / 24 Vdc
- 6 Power supply plug
- 7 Connector
- 8 Bubble level

OPERATING PRINCIPLE

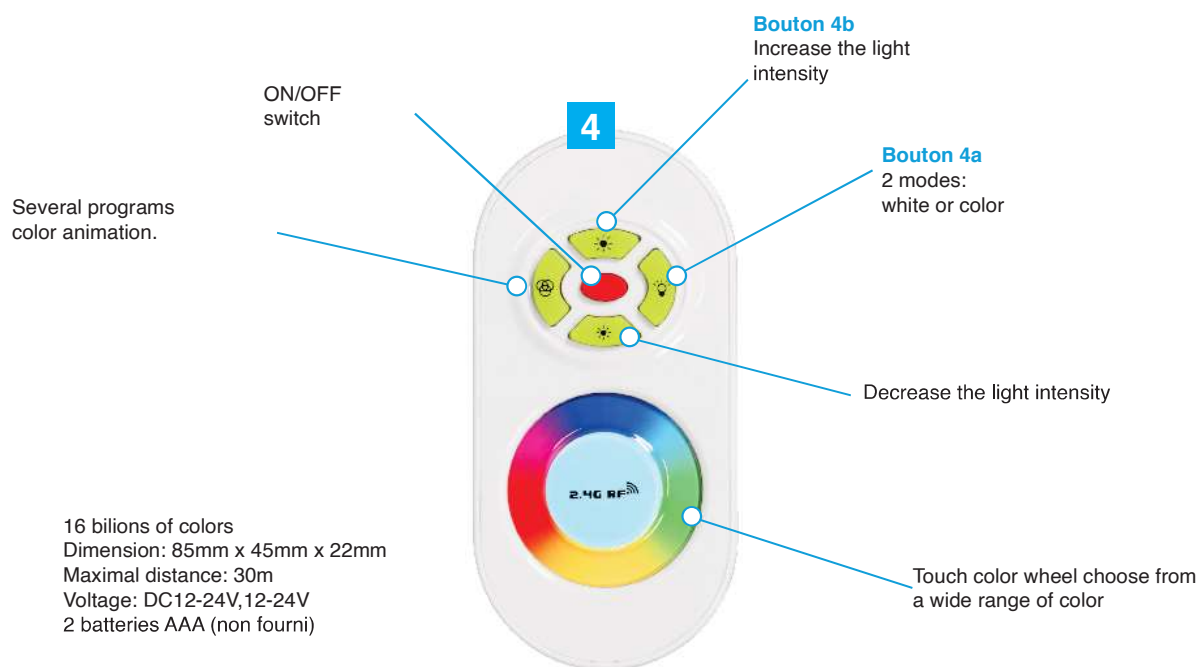
The patient stands on the acrylic part. The mirror under the glass part enables the patient's footprints lit by LED lights to be examined.

CONTENTS OF PACKAGE

- 1 podobaroscope
- 1 power supply 260Vac/24Vac (with power cable).
- 1 user manual
- 1 remote control

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- Remove the different pieces from the packaging.
- Remove the bubble protection of the «U»
- Remove the protective film on the mirror
- Place your Podoscope on a flat, hard and stable floor.
- Check with bubble level **8**
- **Verify the compatibility of your appliance with your mains voltage; the electrical characteristics are on the label affixed to the appliance and at the end of this manual.**
- Connect the power supply plug **6** to the connector **7** of the appliance
- Connect the power source **5** to a mains standard compliant 230 Vac, 50 Hz socket
- Your Podoscope is now ready for use.
- **When installing, check that there is no risk of the power cable getting entangled or being damaged.**

Before using your Podobaroscope ensure you have read the warnings and recommendations on the previous pages.

Switch on the device with the remote control. **4**
Choose the color (white or color) with the **switch 4a**.
Ask the patient to step onto PMMA « U »
Ask the patient to step out PMMA « U »
Switch off the device with the remote control. **4**

4 UPKEEP & MAINTENANCE

For the good working order of your appliance, please follow the upkeep instructions below.

REMOTE CONTROL PAIRING PROCEDURE

To pair the remote control with the device, press **switch 4b** once within 3 seconds when you switch on your PODOBAROSCOPE. The device will flash 3 times to indicate that the remote control is paired.

To reset the remote control, when switching on, press 5 times **switch 4b**. PODOBAROSCOPE will flash 6 times if the reset is validated.

CLEANING

After each use, clean and disinfect the «U» : Septalkan spray ou wipes (Ref. 15410 & 15409).

WARNING : do not use products with more than 10 % of ethanol

Periodically, clean the acrylic glass plate with a very soft cloth slightly dampened with soapy water in order to maintain excellent transparency.

Periodically, clean the mirror with a classic cleaner.

MAINTENANCE & DEFECTS

In the event of damage to the appliance, the transformer, the foot switch or the power cable, the appliance must be examined by our After Sales Service before being used again.

Any part that needs replacing should be a part supplied and validated by our After Sales Service. Any maintenance operations must be carried out with the power off.

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